Therabody

# **THERAGUN RELIEF**

Unit Warnings

## TABLE OF CONTENTS

EN	3-19
ES	20-23
FR	23-27
IT	27-31
DE	31-35
RU	36-40
NL	40-44
ESMX	44-48
FRCA	48-63
SC	64-67
TC	67-70
KR	70-73
JP	74-77

## ThERAGUN RELIEF DEVICE WARNINGS AND GUIDANCE (PRECAUTIONS AND CONTRAINDICATIONS)

### Background

Therabody products are designed to unlock the body's natural ability to achieve health and well-being. Through science and technology, the Therabody portfolio of products allows people to access the therapeutic benefits of different natural phenomena to meet both their needs and preferences. There will be times when it is advisable to modify how devices are used (precautions) or times when it is not appropriate to use certain devices (contraindications). Read the following safety information for the Theragun Relief device in its entirety prior to use.

### IMPORTANT SAFETY INFORMATION

#### General Theragun Relief device Use

Read the full Warnings and Guidance prior to using the Theragun Relief device.

This Theragun Relief device is intended for use by people in good health. This Theragun Relief device is contraindicated against and should not be used by or on anyone with a history of epilepsy, seizures or cardiopathy. Please consult your physician prior to using the Theragun Relief device if you are pregnant and/or nursing. ***Immediately stop using the Theragun Relief device at the first sign of discomfort.***

The Theragun Relief device is NOT a toy. Adult supervision should be provided for those under the age of 18 using

this Theragun Relief device and should be provided by a responsible adult ONLY. If you have any medical considerations please consult your doctor before using the Theragun Relief device.

### SAFETY, PRECAUTIONS AND CONTRAINDICATIONS

#### Specific Therapies

These recommendations are derived from consultation with medical experts and published research regarding precautions and contraindications as of the printing date. For up-to-date information, please visit <https://www.therabody.com/us/en-us/precautions-and-contraindications.html>.

#### Precautions

Due care is required in these circumstances and device use may need to be modified. Consult with a medical professional if you currently have or suspect you may have any of the following conditions or if you have any questions.

- Hypertension (controlled)
- Osteopenia
- Osteoporosis
- Pregnancy
- Diabetes
- Varicose veins
- Bony prominences or regions
- Abnormal sensations (e.g., numbness)
- Sensitivity to pressure
- Recent injury or surgery
- Scoliosis or spinal deformity
- Medications that may alter client sensations

## Contraindications

The following are circumstances where the potential risks may outweigh the benefits. Consult a medical professional before use.

- Skin rash, open wounds, blisters, local tissue inflammation, bruises, infections or tumors
- Deep vein thrombosis, osteomyelitis
- Bone fracture or myositis ossificans
- Hypertension (uncontrolled)
- Acute or severe cardiac, liver, or kidney disease
- Neurologic conditions resulting in loss or altered sensation
- Direct application to the face, throat, or genitalia
- Bleeding disorders
- Recent surgery or injury
- Connective tissue disorders
- Peripheral vascular insufficiency or disease
- Medications that thin the blood or alter sensations
- Direct pressure over surgical site or hardware
- Direct pressure over eyes or throat
- Extreme discomfort or pain
- Severe scoliosis or spinal deformity
- Pacemaker, ICD, or history of embolism
- Allergy to device material (TPU, PC+ABS, aluminum, plastic, PA6, PU foam)

For a full list of contraindications and additional warnings, please visit [www.therabody.com](http://www.therabody.com). Patents & Pending Patents at [www.therabody.com](http://www.therabody.com).

## UNIT WARNINGS

BEFORE USING OR CHARGING THE Theragun Relief

device READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL, ON THE CHARGING CABLE, AND THE Theragun Relief device. If your Theragun Relief device doesn't turn on or the battery indicator displays a low battery level, please charge before first use. The Theragun Relief device is intended for over-the-counter use. The Theragun Relief device is not intended to diagnose, cure, or prevent diseases. Therabody strives to make its devices as safe for intended use as possible. This is an advanced mechanical tool with electric components. If the Theragun Relief device and its accessories are not used or maintained properly, there is a risk of fire, electric shock, or injury. When using the Theragun Relief device, the following basic precautions should always be adhered to:

1. **USE ONLY AS INSTRUCTED.** Use the Theragun Relief device as described in the Theragun User Manual only. Use only Therabody recommended accessories and replacement parts. Do not carry out any maintenance other than as advised by Therabody.
2. **NOT FOR CHILDREN.** The Theragun Relief device and USB-C charging cable are not intended for use by young children or persons with reduced physical, sensory, or reasoning capabilities, or lack of experience and knowledge, unless a responsible adult has given them supervision or instruction. The Theragun Relief device is not to be used as a toy. Do not play with, bend, or pull the electrical components. Advise children not to play with the Theragun Relief device or USB-C charging cable. Adult supervision should be provided for those under 18 using this Theragun Relief device.
3. **CHARGING LOCATIONS.** Charge the Theragun Relief device with a USB-C charging cable. The Theragun Relief device should be charged indoors in a well-ventilated, dry location. Do not charge the Theragun Relief device outdoors, in

a bathroom, or within 10 feet (3.1 meters) of a bathtub, shower, or pool. Do not use the Theragun Relief device or charger on wet surfaces, and do not expose the charger to moisture, rain, or snow. Do not use the Theragun Relief device or its compatible charger in the presence of explosive atmospheres (gaseous fumes, dust, or flammable materials). Sparks may be generated, possibly causing a fire.

4. **DO NOT OVERCHARGE.** Do not leave the Theragun Relief device connected to the USB-C charging cable for more than one hour after the battery has been fully charged. The battery includes a protection system to avoid the risk of overcharging. However, overcharging may reduce its life over time.

5. **DO NOT CRUSH, DROP, OR DAMAGE** The Theragun Relief device **OR CHARGER.** Do not use a USB-C charging cable that has received a sharp blow, been dropped, run over, or damaged in any way.

6. **BATTERY CHEMICALS CAUSE SERIOUS BURNS.** Never allow the internal battery to come into contact with the skin, eyes, or mouth. If a damaged battery leaks chemicals, use rubber or neoprene gloves to dispose of it. If skin is exposed to battery fluids, wash with soap and water and rinse with vinegar. If eyes are exposed to battery chemicals, immediately flush with water for 20 minutes and seek medical attention. Remove and dispose of contaminated clothing.

7. **DO NOT SHORT CIRCUIT.** A battery will short circuit if a metal object makes a connection between the positive and negative contacts on the battery or the 16V connector. Do not place a battery near anything that may cause a short circuit, such as coins, keys, or nails in your pocket. A short-circuited battery may cause fire and personal injury.

8. **DEVICE DISPOSAL.** This Theragun Relief device contains a lithium-ion battery, and care must be taken upon disposal of the Theragun Relief device. Before disposal of this Theragun Relief device, please review your local laws and requirements

surrounding Lithium-Ion Battery disposal. The preferred method of disposal is recycling the whole device.

9. **SERVICE.** If the Theragun Relief device is not working properly, has received a sharp blow, or has been dropped, damaged, left outdoors, or dropped into water, then do not use it. Do not attempt to repair or disassemble the Theragun Relief device which may result in an electric shock or fire.

10. **USAGE.** Discontinue use of the Theragun Relief device if you are feeling pain or discomfort. If at any point during the treatment you feel pain or discomfort, stop the treatment immediately. Discontinue use of the Theragun Relief device if it overheats or becomes uncomfortably hot. Do not place your finger between the attachment and the end of the arm, as this may cause pinching and injury.

11. **USB-C CABLE CARE.** Unplug the USB-C charging cable when not in use. Pull the plug, not the cable, to reduce the risk of damage to the electrical plug and cable. Store cable to ensure it is not stepped on, tripped over, or otherwise subjected to damage or stress. Keep the cable away from heated surfaces, oil, and sharp edges. Never operate this appliance if it has a damaged cable or plug if it is not working properly, if it has been dropped or damaged, or dropped into water. Do not stretch the charger cable or place the cable under strain. Do not handle the cable with wet hands. For long-term storage, store with a fully charged battery. Therabody is not responsible for damages that may occur due to the use of third-party chargers.

12. **DO NOT OPERATE UNDER BLANKET AND PILLOW.** Excessive heating can occur and cause fire, electric shock, or injury.

13. **CHARGING.** There is only one correct USB-C insertion position on the Theragun Relief device. Do not force the cable into place. The Therabody logo should be facing upright when inserted. Fully charge the Theragun Relief device at least once

every six months to prevent device and battery damage.

14. **STORING THE Theragun Relief device AND CHARGING CABLE.** Store in a cool, dry place. Only charge the Theragun Relief device when the ambient temperature is between 0°C/32°F and 40°C/104°F. Do not store the Theragun Relief device, or USB-C charging cable where temperatures may exceed 40°C/104°F, such as in direct sunlight, in a vehicle, or in a metal building during the summer.

15. **DEVICE CARE.** The Theragun Relief device is NOT waterproof. The Theragun Relief device is not machine washable. Do not place or store the Theragun Relief device where it can fall or be pulled into a tub or sink. Do not place in or drop into water or other liquid. Do not reach for an appliance that has fallen into or come into contact with water. Unplug immediately. Clean the Theragun Relief device according to the instructions found in the "Care and Cleaning" section above.

16. **DO NOT DISASSEMBLE.** Disassembly or incorrect reassembly may result in the risk of electric shock, fire, or exposure to battery chemicals. The warranty will be void if the Theragun Relief device, batteries, or charger are disassembled or if any parts have been removed.

### LIMITED WARRANTY

For full warranty information, please visit [www.therabody.com/warranty](http://www.therabody.com/warranty).

Legal Manufacturer  
Therabody  
6100 Wilshire Blvd., Suite 200  
Los Angeles, CA 90048

Please note, this is not a return address or a retail location. No products or packages will be accepted at this location.

Customers who are in need of product support should visit <https://www.therabody.com/us/en-us/support/support.html> for the available contact methods or <https://www.therabody.com/anz/en-au/support/support.html>

FDA-Registered  
Limited Warranty Only With Authorized Retailer Purchase

© 2023 Therabody, Inc. All Rights Reserved.  
Patents at [www.therabody.com/patents](http://www.therabody.com/patents)

Imported and Distributed by:  
Therabody International Limited, 5th floor, 40 Mespil Road,  
DO4C2N4 Dublin, Ireland  
Therabody UK Limited, Portsoken House, 155-157 The Minories,  
London, United Kingdom, EC3N 1LJ

Therabody Australia Pty Ltd, Waterman Business Centre, Suite  
16, C307, Level 3, Ringwood Vic 3134, Australia  
[australia@therabodycorp.com](mailto:australia@therabodycorp.com)

CAN ICES-003(B)/NMB-003(B)  
This device complies with Part 15 of the FCC Rules.  
Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Made in China

Please follow your local  
council's guidelines for  
disposal of electrical waste. 

For Australian Market Only: The purchase of this device is accompanied by a limited warranty. For complete details about the rights afforded to you under this warranty, visit <https://www.thebody.com/anz/en-au/warranty-support/>.

### Label & Symbols The following labels and symbols appear on Theragun RELIEF

No.	Icon	Description
1		CE Mark
2		Separate collection for waste electrical and electronic equipment. Note: For more information about disposal of equipment, its parts and accessories, please contact your local distributor.
3	<b>IP22</b>	IP Classification: The first number 2: Protected against access to hazardous parts with a finger, and the jointed test finger of 12 mm $\Phi$ , 80 mm length, shall have adequate clearance from hazardous parts, and protected against solid foreign objects of 12.5 mm $\Phi$ and greater.
4		Please read the user manual before use.
5		Type BF apply part
6		Manufacturer information
7	<b>UK CA</b>	UKCA Mark for UK

Issue	Solution	Corrective Action
Device turns off automatically or Device does not turn on	<ol style="list-style-type: none"><li>1. Battery level is low</li><li>2. Device still does not turn on</li></ol>	<ol style="list-style-type: none"><li>1. Charge the battery</li><li>2. Contact Therabody</li></ol>
Device shuts off mid-treatment or Vibration motor does not turn on	<ol style="list-style-type: none"><li>1. Check and recharge batteries.</li><li>2. Motor or connection have been damaged</li></ol>	Try charging the device fully, removing the charging cable and then pressing left button for 3 seconds. If problem persists, contact Therabody
Battery will not charge	<ol style="list-style-type: none"><li>1. The ambient temperature is too high or too low</li><li>2. Possible defective charging cable</li><li>3. Possible defective battery</li></ol>	<ol style="list-style-type: none"><li>1. Charge the device when the ambient temperature is between 5° C and 35° C</li><li>2. Try using a different USB-C cable and adapter</li><li>3. Contact Therabody</li></ol>

This device is type BF applied equipment. It complies with Medical Electrical Safety Standards (IEC60601-1). This device also complies with Medical EMC Standard (IEC 60601-1-2). The has been tested and found to comply with the electromagnetic compatibility (EMC) limits for medical devices to IEC 60601-1-2: 2014. These limits are designed to provide reasonable protection against harmful interference in atypical medical installation.

**WARNING:**

1\* **WARNING:** Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment

should be observed to verify that they are operating normally.

2\* **WARNING:** Use of accessories, transducers and cables other than those specified or provided by the manufacturer this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in proper operation.

3\* **WARNING:** Portable RF communications equipment (including peripherals such as antenna cables) should be used no closer than 30 cm (12 inches) to any part of the ME EQUIPMENT, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result

**Guidance and manufacturer's declaration – electromagnetic emission – for all EQUIPMENT AND SYSTEMS**

<b>Guidance and manufacturer's declaration – electromagnetic emission</b>		
<b>The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of Theragun RELIEF should assure that it is used in such an environment.</b>		
<b>Emissions test</b>	<b>Compliance</b>	<b>Electromagnetic environment - guidance</b>
RF emissions  CISPR 11	Group 1	The Theragun RELIEF uses RF energy only for its internal function. There for, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

## Guidance and manufacturer's declaration – electromagnetic emission

**The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of Theragun RELIEF should assure that it is used in such an environment.**

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Class B	The Theragun RELIEF suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations flicker emissions IEC 61000-3-3	Not applicable	

**Guidance and manufacturer's declaration – electromagnetic immunity**

**The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of the Theragun RELIEF should assure that it is used in such an environment.**

<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment –guidance</b>
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	±8 kV contact ±2 kV, ±4 kV, ±8 kV, ±15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	N/A	Mains power quality should be that of a typical commercial or home environment.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	N/A	Mains power quality should be that of a typical commercial or home environment.

## Guidance and manufacturer's declaration – electromagnetic immunity

**The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of the Theragun RELIEF should assure that it is used in such an environment.**

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	0 % UT; 0,5 cycle g) At 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315°  0 % UT; 1 cycle and 70 % UT; 25/30 cycles Single phase: at 0°  0 % UT; 250/300 cycle	N/A	Mains power quality should be that of a typical commercial or home environment. If the user of the Theragun RELIEF requires continued operation during power mains interruptions, it is recommended that the Theragun RELIEF be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or home environment.

**NOTE**      UT is the a. c. mains voltage prior to application of the test level.

## Guidance and manufacturer's declaration – electromagnetic immunity

The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of the Theragun RELIEF should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment -guidance
<p>Conducted RF</p> <p>IEC 61000-4-6</p>	<p>3 Vrms</p> <p>150 kHz to 80 MHz</p> <p>6 V in ISM and amateur radio bands between 0,15 MHz and 80 MHz</p>	<p>N/A</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the Theragun RELIEF, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = \left[ \frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[ \frac{12}{V_2} \right] \sqrt{P}$

## Guidance and manufacturer's declaration – electromagnetic immunity

The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of the Theragun RELIEF should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment –guidance
Radiated RF IEC 61000-4-3	10 V/m 80 MHz to 2.7 GHz  385MHz-5785MHz Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication equipment (Refer to table 9 of IEC 60601-1-2:2014)	10 V/m 80 MHz to 2.7 GHz  385MHz-5785MHz Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communication equipment (Refer to table 9 of IEC 60601-1-2:2014)	$d = \left[ \frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[ \frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.7 \text{ GHz}$ <p>where p is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).b</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey,a should be less than the compliance level in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

### Guidance and manufacturer's declaration – electromagnetic immunity

The Theragun RELIEF is intended for use in the electromagnetic environment specified below. The customer or the user of the Theragun RELIEF should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment –guidance
NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic is affected by absorption and reflection from structures, objects and people.			
<p>a The ISM (industrial, scientific and medical) bands between 150 kHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.</p> <p>b Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Theragun RELIEF is used exceeds the applicable RF compliance level above, the Theragun RELIEF should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Theragun RELIEF.</p> <p>c Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3V/m.</p>			

**Recommended separation distances between portable and mobile RF communications equipment and the Theragun RELIEF**

The Theragun RELIEF is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Theragun RELIEF can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Theragun RELIEF as recommended below, according to the maximum output power of the communications equipment

**Separation distance according to frequency of transmitter**  
m

Rated maximum output of transmitter W	150 kHz to 80 MHz outside ISM and amateur radio bands $d = \left[\frac{3.5}{V_1}\right]\sqrt{P}$	150 kHz to 80 MHz in ISM and amateur radio bands $d = \left[\frac{12}{V_2}\right]\sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1}\right]\sqrt{P}$	800 MHz to 2.7 GHz $d = \left[\frac{7}{E_1}\right]\sqrt{P}$
0.01	0.12	0.20	0.035	0.07
0.1	0.38	0.63	0.11	0.22
1	1.2	2.00	0.35	0.70

**Recommended separation distances between portable and mobile RF communications equipment and the Theragun RELIEF**

The Theragun RELIEF is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Theragun RELIEF can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Theragun RELIEF as recommended below, according to the maximum output power of the communications equipment

	Separation distance according to frequency of transmitter m			
Rated maximum output of transmitter W	150 kHz to 80 MHz outside ISM and amateur radio bands $d = \left[ \frac{3.5}{V_1} \right] \sqrt{P}$	150 kHz to 80 MHz in ISM and amateur radio bands $d = \left[ \frac{12}{V_2} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[ \frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.7 GHz $d = \left[ \frac{7}{E_1} \right] \sqrt{P}$
10	3.8	6.32	1.10	2.21
100	12	20.00	35	70

**Recommended separation distances between portable and mobile RF communications equipment and the Theragun RELIEF**

**The Theragun RELIEF is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Theragun RELIEF can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Theragun RELIEF as recommended below, according to the maximum output power of the communications equipment**

**Separation distance according to frequency of transmitter  
m**

For transmitters rated at a maximum output power not listed above the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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© 2023 Therabody, Inc. All Rights Reserved. Patents at [www.therabody.com/patents](http://www.therabody.com/patents)