Therabody

TheraCupTM

Advanced Cupping Therapy

Unit Warnings

Table of contents

EN	2 - 6
DE	7 - 11
FR	12 - 16
IT	17 - 21
ES	22 - 26
SC	27 - 31

TheraCup™ Warnings and Guidance (Precautions and Contraindications)

Background

This device is intended to be used on large muscle groups. If you have any specific medical conditions or concerns, please consult your physician before using this product. At times, it will be advisable to modify how devices are used (precautions) and times when it is not appropriate to use certain devices (contraindications). The following document highlights these for TheraCup™ as of the print date. For up-to-date information, please visit us online at www.therabody.com.

Important Safety Information

General TheraCup Device Use

Read the full Warnings and Guidance prior to using TheraCup™ Device.

This device is intended for use by people in good health. This device is contraindicated for those with acute or severe cardiac, liver, or kidney disease/condition or other underlying conditions linked to these diseases. This device is contraindicated and not intended for use on the face, neck, and head; however, the smallest cup may be used on the levator scapulae muscle with caution. It is not recommended that this device is used over various skin infections and conditions, or directly over a surgical site/hardware, bone fracture, or if you are experiencing unexplained pain in the area. It is also advised that this device is not used for those who have neurological and/or peripheral conditions as well as other conditions that may influence or alter sensation and perception. In the event of any adverse reaction, discontinue use. Please consult your physician prior to use if you are unsure if you have any of the previously listed conditions. ***Immediately stop using the device at the first sign of discomfort. the use of this device, please consult your physician prior to using the device.***

Adult supervision should be provided for those under the age of 18 using this device. If you have any medical concerns regarding the use of this device, please consult your physician prior to using the device.

TheraCup Safety, Precautions, and Contraindications

These recommendations are derived from consultation with medical experts and the published research in regards to precautions and contraindications for myofascial interventions. For up-to-date information, please visit us online at www.therabody.com.

Precautions:

Due care is required in these circumstances and the use of the devices may need to be modified. Where appropriate or if you have any concerns, seek the advice of a medical professional.

- Recent injury or surgery
- Pregnancy
- Diabetes
- Osteopenia
- Osteoporosis
- Hypertension (controlled)
- Abnormal sensations (e.g., numbness)
- Bony prominences or regions
- Sensitivity to heat, vibration, or pressure
- Mild peripheral neuropathy
- Do not wear this device overnight while sleeping

Contraindications:

The following are circumstances where the potential risks may outweigh the benefits. Consult a medical professional before use.

- Skin rash, open wounds, blisters, local tissue inflammation, infections, bruises, or tumors
- Skin conditions such as eczema or psoriasis
- Bone fracture or myositis ossificans
- Osteomyelitis
- Severe scoliosis or spinal deformity
- Hypertension (uncontrolled)
- Vascular/arterial and circulation diseases, including blood clots, phlebitis, thromboses (DVT), and varicose veins
- History of embolism
- Acute or severe cardiac, liver, or kidney disease
- Severe peripheral neuropathy or other cause of sensory impairment
- Bleeding disorders
- Connective tissue disorders
- Medications that thin the blood or alter sensations
- Direct pressure over surgical site or hardware
- Direct application to the face, head, throat, neck, or genitalia
- Pacemaker or ICD
- Do not use in the presence of unexplained pain

LIMITED WARRANTY

For full warranty information, please visit www.therabody.com/warranty. To request a copy of the warranty by mail, you may send a request to the following address:

Therabody - Warranty
Attn: Customer Service
6100 Wilshire Blvd., Suite 200
Los Angeles, CA 90048

Please note, this is not a return address or a retail location. No products or packages will be accepted at this location.

Additional Unit Warnings

UNIT WARNINGS

BEFORE USING OR CHARGING TheraCup™, READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL, ON THE CHARGER, ON THE BATTERY, AND ON TheraCup™. TheraCup™ is intended for commercial or home use.

TheraCup™ is not intended to diagnose, or prevent any diseases. Therabody strives to make TheraCup™ as safe for intended use as possible. TheraCup™ is an advanced mechanical tool with electric components. If TheraCup™ and its accessories are not used or maintained properly, there is a risk of fire, electric shock, or injury. When using TheraCup™, the following basic precautions should always be adhered to:

1. **USE ONLY AS INSTRUCTED.** Use TheraCup™ as described in the Instruction Manual only. Use only Therabody recommended accessories and replacement parts. Do not carry out any maintenance other than as advised by Therabody.

2. **NOT FOR CHILDREN.** TheraCup™ and charger are not intended for use by young children or persons with reduced physical, sensory, or reasoning capabilities, or lack of experience and knowledge, unless a responsible adult has given them supervision or instruction. TheraCup™ is not to be used as a toy. Advise children not to play with TheraCup™ or charger. Adult supervision should be provided for those under 18 while using this device.

3. **CHARGING LOCATIONS.** Before charging, toggle the MasterSwitch into the OFF position completely after the treatment (Battery LED OFF). Do not use TheraCup™ while charging. Avoid skin contact with the connector when the USB-C to USB-A Cable is plugged into a power source as it may cause discomfort or injury. Do not sleep or sit on the USB-C connector. Charge TheraCup™ with a USB-C charger. TheraCup™ should be charged indoors in a well-ventilated, dry location. Do not charge TheraCup™ outdoors, in a bathroom, or within 10 feet (3.1 meters) of a bathtub or pool. Do not use TheraCup™ or charger on wet surfaces, and do not expose the charger to moisture, rain, or snow. Do not use TheraCup™ or its compatible charger in the presence of explosive atmospheres (gaseous fumes, dust, grain, metal powders, or flammable materials). Sparks may be generated, possibly causing a fire. Obey all signs and instructions.

4. **UNPLUGGING THE CHARGER.** Do not overcharge TheraCup™. Pull the plug, not the cord, to reduce the risk of damage to the electrical plug and cord. Never carry the charger by its cord. Keep the cord away from heat, oil, and sharp edges. Do not use or charge with a damaged cord. Do not stretch the charger cord or place the cord under strain. Keep the cord away from heated surfaces. Do not handle the charger, terminals, or TheraCup™ with wet hands. Always unplug this appliance from the electrical outlet immediately after using and before cleaning.

5. **THERABODY USB-C CHARGING CABLE CARE.** Unplug the adapter when not in use. Store cord away so that it will not be stepped on, tripped over, or otherwise subjected to damage or stress. Replace a damaged charger. Keep the cord away from heated surfaces. Never operate this appliance if it has a damaged cord or plug if it is not working properly, if it has been dropped or damaged, or dropped into water. For long-term storage, keep the battery fully charged. Therabody is not responsible for damages that may occur due to the use of third-party charging cables.

6. **BATTERY.** Do not attempt to replace the TheraCup™ battery yourself—damaging the battery may cause overheating and injury.

7. **USAGE.** Discontinue the use of TheraCup™ if you are feeling severe pain. Do not use TheraCup™ on the face, head, throat, neck, or genitals. Only use TheraCup™ on hydrated skin. Feel the suction setting on your arm and adjust the desired setting accordingly before starting the device.

8. **CLEANING INSTRUCTIONS.** Before cleaning, toggle the MasterSwitch to the left to turn OFF TheraCup™. Ensure that the battery status LED is OFF. Use disinfectant wipes for sanitizing the cups and heating element. Wipe the device (including the Heat Metal at the bottom) and cups clean with a non-abrasive, lint-free cloth. Lightly dampen the cloth with fresh water or an alcohol-based solution if necessary. Dry TheraCup™ with a non-abrasive, lint-free cloth. Clean TheraCup™ with a lightly dampened cloth or an alcohol-based solution if it comes in contact with anything that may cause stains or other damage, like: dirt or sand, makeup, ink, soap, detergent, acids or acidic foods, or with liquids other than fresh water, including those that may lead to skin irritation such as: sweat,

salt water, soapy water, pool water, perfume, insect repellent, lotions, sunscreen, oil, adhesive remover, hair dye, or solvents. Dry TheraCup™ with a non-abrasive, lint-free cloth. TheraCup™ comes with replaceable filters to prevent foreign substances from damaging the device. Replace the filter once every six months.

9. DEVICE CARE. Handle TheraCup™ with care. TheraCup™ contains electronic components that can be damaged if dropped, burned, punctured, or crushed. Do not disassemble TheraCup™ or attempt to repair TheraCup™ yourself. Disassembling TheraCup™ may damage it, result in loss of water resistance, and may cause injury. If TheraCup™ is damaged or if malfunctions occur, contact Therabody Customer service. Do not use a damaged TheraCup™, for example, visible water intrusion, as these may cause injury. Avoid heavy exposure to dust or sand.

SAVE THESE INSTRUCTIONS

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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