THERAGUN mini

Table of contents

EN	3-8
ES	9-15
FR	16-22
IT	23-29
DE	30-36
RU	37-43
ESMX	44-50
FRCA	51-57
SC	58-65
TC	66-73
KR	74-80
JP	81-87

Getting started

Step 1

Turn on and float

To turn on the Theragun mini, firmly press and hold the multi-functional power/speed button on the side of the device, then float the Theragun mini across the areas in need. Apply light to moderate pressure as needed.

Step 2

Get the Therabody app


We'll guide you through a stress-free setup and show you how to get the most out of your Theragun mini.

Step 3

Feel the difference.™ Every day.

Take a deep breath and treat yourself with guided treatment presets or freestyle mode. It's all designed for you.

See warnings insert for important safety instructions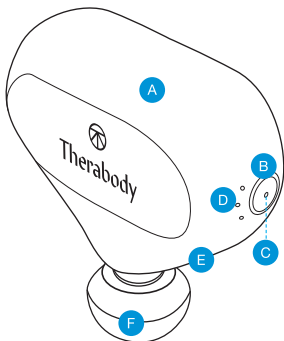

- A Ergonomic grip
- B Multi-functional power/speed button
- C LED battery indicator light
- D LED speed indicator
- E USB-C charging port
- F Interchangeable attachment connector


Using the Theragun mini

1. To turn on the Theragun mini, firmly press and hold the multi-functional power/speed button on the right side of the device.
2. Float the device across areas in need of treatment, gradually applying light to moderate pressure as needed.
3. Press the power/speed button again to toggle between 3 speeds.
4. To turn off the Theragun mini, firmly press and hold the power/speed button until the device shuts off.

Connecting the attachments

1. Ensure that your device is powered off before connecting or removing any attachments.
2. Align the attachment with the tip of the arm and push to connect.
3. To remove the attachment, hold the top of the device with one hand, and the attachment with the other hand. Place one finger on each side of the plastic connector and firmly pull off the attachment.

Warning: Do not place your finger between the attachment and the end of the arm, as this may cause pinching or injury.

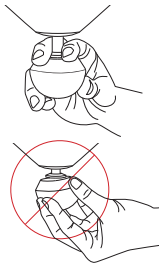EN Charging the Theragun mini

1. The Theragun mini is USB-C enabled. For charging, after turning off the device, plug one end of the cable included in the package into your USB of choice and the other end to the device.
2. The LED light on the power/speed button will flash until fully charged.
3. Charging is complete when the LED light on the power/speed button is solid green.
4. Battery level is continuously displayed on the LED light of the power/speed button when the device is on.
5. To view battery level when the device is off, briefly press the power/speed button and the LED battery light will flash to indicate the battery level.

Battery Level

Battery level on the Theragun mini is indicated with the following colors:

-  **Flashing orange:** Low charge
-  **Flashing blue:** Half-way charged
-  **Flashing green:** High charge
-  **Solid green:** Fully charged

Cleaning the Theragun mini

For the most hygienic Theragun device experience, sanitize it after each use by:

1. Wiping the device with a disinfectant wipe to remove any residue.
2. Once the device is residue-free, use a new wipe to disinfect the surface area by wiping all sides in a downward motion.

Smart Features

Your Theragun mini is a smart device enabled with Bluetooth connectivity, designed to pair seamlessly with the Therabody app. When you download the app, you'll be guided through a step-by-step setup and shown how to pair your device.

Once paired with your device, the app can remotely adjust speed, activate guided treatment presets, and make real-time recommendations based on your usage.

Scan the QR code
to get started.

Attachments

Your Theragun mini comes with 3 attachments designed for a more customized treatment.

- Made of comfortable closed-cell PU foam
- Easy to clean for a hygienic experience
- Won't absorb sweat, lotions, or oils

**Standard
Ball**

A more targeted option for overall use. The Standard Ball attachment is specifically designed for overall use on both large and small muscle groups. With a smaller shape and higher impact level than the Dampener, it is a more targeted option for overall use and is your go-to attachment during a workout.

Dampener

For tender or bony areas and overall use. The Dampener attachment is specifically designed for general use and low impact treatment around tender or bony areas. Its wide base maximizes surface area for greater, more efficient contact with the body.

Thumb

For trigger points and the lower back. Specifically designed to mimic an actual human thumb, the Thumb attachment is perfect for trigger point therapy and use on the lower back. Use the Thumb anywhere you might have used an actual thumb and get the treatment you need without needing to leave the comfort of home.

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