Therabody Lounger

Unit Warnings

Table of contents

EN	3 - 7	FR	13 - 17	ES	23 - 27
DE	8 - 12	IT	18 - 22		

Therabody Lounger Warnings and Guidance (Precautions and Contraindications)

Background

Therabody Lounger is intended to be used while sitting or lying down. If you have any specific medical conditions or concerns, please consult your physician before using this product. At times, it will be advisable to modify how products are used (precautions) and times when it is not appropriate to use certain products (contraindications). The following document highlights the precautions and contraindications for Therabody Lounger as of the print date. For up-to-date information, please visit us online at <https://www.therabody.com/us/en-us/precautions-and-contraindications.html>.

Important Safety Information

General Therabody Lounger Use

Read the full Warnings and Guidance prior to using Therabody Lounger. This product is intended for use by people in good health. This product is contraindicated against and should not be used by or on anyone with an electronic implanted device (such as a pacemaker), cardiac arrhythmia, tumors, or acute episodes of inflammatory diseases. This product is contraindicated against and should not be used by those with a history of seizures or acute or severe cardiac, liver, or kidney disease/condition or other underlying conditions linked to these diseases. It is also advised that this product is not used for those who have neurological and/or peripheral conditions or other conditions that may affect or alter sensation and perception. In the event of any adverse reaction, discontinue use. Please consult your physician prior to use if you are unsure if you have any of the previously listed conditions. ***Immediately stop using the product at the first sign of discomfort.***

Adult supervision should be provided for those under 18 using Therabody Lounger. If you have any medical concerns regarding using this product, please consult your physician before using Therabody Lounger.

Therabody Lounger Safety, Precautions, and Contraindications

These recommendations are derived from consultation with medical experts and the published research regarding precautions and contraindications for Vibroacoustic therapy. For up-to-date information, please visit us online at <https://www.therabody.com/us/en-us/precautions-and-contraindications.html>.

Precautions:

Due care is required in these circumstances, and the use of the product may need to be modified. Where appropriate, or if you have any concerns, seek the advice of a medical professional.

- Hypertension (controlled)
- Diabetes
- Bony prominences or regions
- Recent injury or surgery
- Scoliosis or spinal deformity
- Abnormal sensations (e.g., numbness)
- Medications that may alter sensations

Contraindications:

The following are circumstances where the potential risks may outweigh the benefits. Consult a medical professional before use.

- Open wounds, infections, or tumors
- Acute inflammatory disease
- Pregnancy
- Hypertension (uncontrolled)
- Hypotension
- Psychoses
- Seizure disorders
- Vascular/arterial and circulation diseases, including blood clots, phlebitis, thromboses, and varicose veins
- Pacemaker or ICD
- History of embolism

- Acute or severe cardiac, liver, or kidney disease
- Neurologic conditions resulting in loss or altered sensation
- Bleeding disorders
- Connective tissue disorders
- Medications that thin the blood or alter sensations
- Direct pressure over surgical site or hardware
- Extreme discomfort or pain

ADDITIONAL UNIT WARNINGS LIMITED WARRANTY

For full warranty information, please visit www.therabody.com/warranty. To request a copy of the warranty by mail, you may send a request to the following address:

Therabody - Warranty
Attn: Customer Service
6100 Wilshire Blvd, Suite 200
Los Angeles, CA 90048

Please note, this is not a return address or a retail location. No products or packages will be accepted at this location.

©2022 Therabody, Inc. All Rights Reserved. ®USA & China

Patents & Pending Patents at www.therabody.com/patents

UNIT WARNINGS

BEFORE USING THERABODY LOUNGER, READ ALL INSTRUCTIONS AND CAUTIONARY MARKINGS IN THIS MANUAL, ON THE ADAPTER, AND THERABODY LOUNGER BEFORE USE. Turn the volume and intensity dials to zero before turning on Therabody Lounger. Therabody Lounger is intended for home and commercial use.

Therabody Lounger is not intended to diagnose, mitigate, or treat diseases. Therabody strives to make Therabody Lounger as safe for intended use as possible. Therabody Lounger is an advanced mechanical tool with electric components. If Therabody Lounger and its accessories are not used or maintained properly, there is a risk of fire, electric shock, or injury. When using Therabody Lounger, the following basic precautions should always be adhered to:

- **Danger:** Always unplug Therabody Lounger from the electrical outlet immediately after using and before cleaning or

servicing to reduce the risk of electrical shock.

- Warning: Risk of electric shock, fire, and injury. This product must be used according to the following instructions.

Important safety instructions when using an electrical appliance, basic precautions should always be followed, including the following: read all instructions before using this appliance.

1. **USE ONLY AS INSTRUCTED.** Use Therabody Lounger as described in this Therabody Lounger Instruction Manual only. Use only Therabody recommended accessories and replacement parts. Do not carry out any maintenance other than as advised by Therabody.
2. **NOT FOR CHILDREN.** Therabody Lounger and adapter are not intended for use by young children or persons with reduced physical, sensory, or reasoning capabilities or lack of experience and knowledge unless a responsible adult has given them supervision or instruction. Therabody Lounger is not to be used as a toy. Advise children not to play with Therabody Lounger or adapter. Keep children away from extended foot support (or other similar parts). Adult supervision should be provided for those under 18 while using this product.
3. **USAGE LOCATIONS.** The power adapter only fits into a polarized outlet one way. If the plug does not fit fully into the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to force the plug. Avoid skin contact with the power adapter when it is plugged into a power source as it may cause discomfort or injury. Do not sleep or sit on the power adapter. Therabody Lounger should be used indoors in a well-ventilated, dry location. Do not use Therabody Lounger outdoors, in a bathroom, or within 10 feet (3.1 meters) of a bathtub or pool. Do not use Therabody Lounger or adapter on wet surfaces, and do not expose the adapter to moisture, rain, or snow. Do not use Therabody Lounger or its compatible cord in the presence of explosive atmospheres (gaseous fumes, dust, grain, metal powders, or flammable materials). Sparks may be generated, possibly causing a fire. Obey all signs and instructions. After usage, turn all controls into the off position and unplug the power cord from the outlet.
4. **UNPLUGGING THE ADAPTER.** Pull the plug, not the cord, to reduce the risk of damage to the electrical plug and cord. Never carry by the cord. Keep the cord away from heat, oil, and sharp edges. Do not use a damaged cord. Replace a damaged cord by contacting the manufacturer, a service agent, or someone equally qualified to avoid a hazard. Do not stretch the cord or place the cord under strain. Keep the cord away from heated surfaces. Do not handle the adapter, terminals, or Therabody Lounger with wet hands. Therabody Lounger should never be left unattended when plugged in. Unplug Therabody Lounger from the outlet when not in use and when putting on or taking off parts. Always unplug this appliance before cleaning.
5. **THERABODY CORD CARE.** Store cord away so that it will not be stepped on, tripped over, or otherwise subjected to damage or stress. Replace a damaged cord. Keep the cord away from heated surfaces. Never operate this appliance if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water. Therabody is not responsible for damages that may occur due to the use of third-party adapters.
6. **USAGE.** Discontinue use of Therabody Lounger if you are feeling abnormal discomfort or pain while operating the product. Consult your physician before using Therabody Lounger again. Do not stand on or in Therabody Lounger. Use Therabody Lounger only while seated. Do not place your hands or fingers near the footrest mechanism on the chair's underside while the product is on. Touching the mechanism can result in injury. Never operate Therabody Lounger with the air openings blocked. Keep the air openings free of lint, hair, etc. Never drop or insert objects into any opening. Do not use outdoors. Do not operate in the presence of aerosol (spray) or where oxygen is being administered. Comfortable clothing is recommended while using

this product. Always confirm there are no obstacles behind Therabody Lounger with sufficient space to recline. Do not use Therabody Lounger when standing on a damp floor or are in contact with any plumbing. To avoid injury, keep pets away from the chair when in use.

7. **PRODUCT CARE.** Handle Therabody Lounger with care. If Therabody Lounger has been in storage, confirm that the product operates properly before use. Do not use Therabody Lounger if noises above normal sound occur. Therabody Lounger contains electronic components that can be damaged if dropped, burned, punctured, or crushed. Do not disassemble the SmartGoggles or attempt to repair Therabody Lounger yourself. Disassembling Therabody Lounger may damage it, result in loss of water resistance, and may cause injury. Do not use damaged Therabody Lounger, for sample, visible water intrusion, or damage, as these may cause injury. Avoid heavy exposure to dust or sand. If Therabody Lounger is damaged or if malfunctions occur, contact Therabody Customer service.

8. **CLEANING INSTRUCTIONS.** Clean Therabody Lounger with a damp cloth if they come in contact with anything that may cause stains, or other damage, such as dirt or sand, makeup, ink, soap, detergent, acids or acidic foods, or with liquids other than water, including those that may lead to skin irritation such as: sweat, salt water, soapy water, pool water, perfume, insect repellent, lotions, sunscreen, oil, adhesive remover, hair dye, or solvents.

SAVE THESE INSTRUCTIONS

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

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©2022 Therabody, Inc. Todos los derechos reservados. ® EE. UU. & China