


THERAGUN mini

The Theragun mini™

See warnings insert for important safety instructions

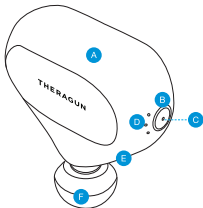

- A Ergonomic grip
- B Multi-functional power/ speed button
- C LED battery indicator light
- D LED speed indicator
- E Charging port
- F Interchangeable attachment connector

Using the Theragun mini

1. To turn on the Theragun mini, firmly press and hold the multi-functional power/ speed button on the right side of the device.
2. Float the device across areas in need of treatment, gradually applying light to moderate pressure as needed.
3. Press the power/speed button again to toggle between 3 speeds.
4. To turn off the Theragun mini, firmly press and hold the power/speed button until the device shuts off.

Connecting the attachment

Ensure that your device is powered off before connecting or removing any attachments. Align the attachment with the tip of the arm and push to connect. To remove the attachment, hold the top of the device with one hand, and the attachment with the other hand. Place one finger on each side of the plastic connector and firmly pull off the attachment. **Warning: Do not place your finger between the attachment and the end of the arm, as this may cause pinching or injury.**







Your Theragun mini comes with a Standard Ball attachment designed for overall use. It is also compatible with all available attachments, excluding those exclusive to the first-generation Theragun liv.

Charging the Theragun mini

1. After turning off the device, plug the power adapter into the charging port of the Theragun mini.
2. The LED light on the power/speed button will flash until fully charged.
3. Charging is complete when the LED light on the power/speed button is solid green.
4. Battery level is continuously displayed on the LED light of the power/speed button when the device is on. To view battery level when the device is off, briefly press the power/speed button and the LED battery light will flash to indicate the battery level.

Battery Level

Battery level on the Theragun mini is indicated with the following colors:

-  **Flashing red:** Low charge
-  **Flashing blue:** Half-way charged
-  **Flashing green:** High charge
-  **Solid green:** Fully charged

Cleaning the Theragun mini

For the most hygienic Theragun device experience, sanitize it after each use by:

1. Wiping the device with a disinfectant wipe to remove any residue.
2. Once the device is residue-free, use a new wipe to disinfect the surface area by wiping all sides in a downward motion.