

RecoveryAir PRO

RecoveryAir™ PRO Pneumatic Device
Quick Start Guide

Distributed by: Theragun Inc.,
1515 W. Wardlow Rd. Long Beach, CA 90810
www.therabody.com



Scan the QR code to get started.

Pour commencer, numérisiez le code QR.

Para comenzar, escanee el código de respuesta rápida (Quick Response, QR).

Scannen Sie den QR-Code und legen Sie los.

Scanner le QR code pour démarrer.

Effettua la scansione del codice QR per cominciare.

Escanee el código QR para empezar.

Scan de QR-code om aan de slag te gaan.

Zeskanuj kod QR, aby rozpocząć.

Σαρώστε τον κωδικό QR για να ξεκινήσετε.

Skann QR-koden for å komme i gang.

Skenirajte QR kód da biste počeli.

Skanna QR-koden för att komma igång.

QR kodunu tarayarak başlayın.

Realize a leitura do código QR para começar.

Отсканируйте QR-код, чтобы начать.

امسح رمز الاستجابة السريعة للبدء.

掃描二維碼 開始。

掃描 QR 碼 開始使用。

시작하시려면 QR 코드를 스캔하십시오.

QRコードをスキャンして 始めましょう。

Get the most out of your RecoveryAir PRO pneumatic device

See Warnings insert for important safety instructions

A Mode/Menu button
Used to access the selection of a program, treatment pressure, treatment time, cycle mode, release time, and hold time.

B Up and Down buttons
Used to select a program, treatment pressure, treatment time, cycle mode, release time, and hold time.


C Start/Stop button
Used to start or stop treatment.

D Power Switch
Used to turn On/Off the RecoveryAir PRO Pneumatic Device.

E DC Adapter Socket
Used for connecting the DC power adapter.

F LCD Display Screen

- Displays pressure setting at the most distal pressure zone.
- Pressure may be increased or decreased from this home screen.
- Displays treatment time setting, which decreases during treatment.

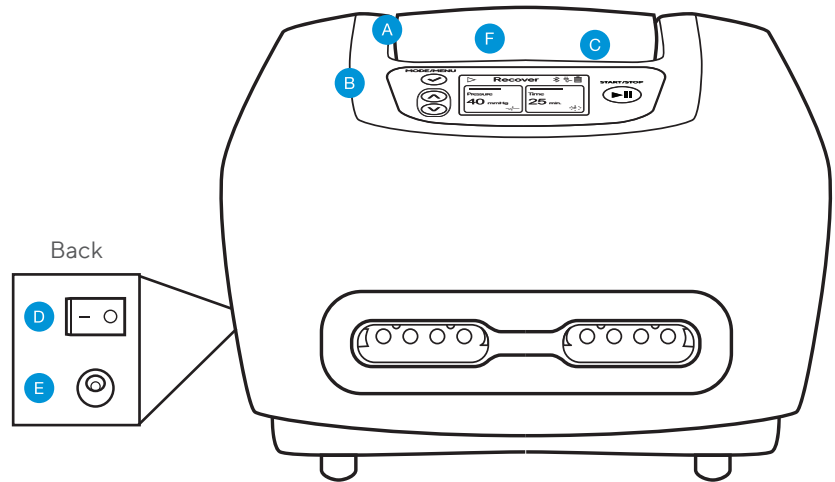

Using the RecoveryAir PRO Pneumatic Device







1. Place the RecoveryAir PRO Pneumatic Device on a flat stable surface and insert the DC power adapter cable into the DC adapter socket on the device.
2. Put on the garment.
3. Connect the garment hoses to the RecoveryAir PRO Pneumatic Device by holding the device handle with one hand while connecting or disconnecting the hose bundle with your other hand.
4. Insert the Blocker Plug to block the unused air outlet for single garment use.
5. Select the Treatment Program.
 - From the Home screen, press (✓) to select the desired program.
6. Set Treatment Pressure.
 - Press (✓) repeatedly until the desired program screen appears.
 - Press and hold (✓) until the Pressure value screen is displayed.
 - Press (✓) until the highlighted pressure value blinks.
 - To select Pressure, press (▲) or (▼).
 - To confirm a selection, press (✓).
7. Set Treatment Time
 - Press (✓) until the highlighted Time value blinks.
 - To select Time, press (▲) or (▼).
 - To confirm a selection, press (✓).

Using the RecoveryAir PRO Pneumatic Device (continued)

8. Set Hold Time
 - Press (✓) until the highlighted Hold value blinks.
 - To select Hold, press (▲) or (▼).
 - To confirm a selection, press (✓).
9. Set Release Time
 - Press (✓) until the highlighted Release value blinks.
 - To select Release, press (▲) or (▼).
 - To confirm a selection, press (✓).
10. Set Gradient
 - Press (▼) and press (✓) on the word Gradient.
 - Press (✓) until the selected zone blinks.
 - To select the specific zone pressure, press (▲) or (▼).
 - To confirm a selection, press (✓).
 - Repeat the above steps for all zones.
11. Start the treatment

Charging the RecoveryAir PRO Pneumatic Device

- Turn On the Power Switch to enable charging the battery.
- Turn Off the Power Switch to conserve battery power when not charging or not using the RecoveryAir PRO Pneumatic Device.
- Battery level is continuously displayed by the battery status indicators.

Battery Status	Displayed Icon	Available Power
		Constant bars 100%
		Constant bars 75%
		Constant bars 50%
		Constant bars 25%
		Blinking bar Almost empty
		No bars Empty

Using your mobile app via Bluetooth connectivity

- Modify treatment parameters
 - Cycle mode
 - Treatment time
 - Treatment pressure
 - Hold time
 - Release time
 - Zone pressures
- Retrieve usage history from the device
- Start or stop the device using your smartphone